

# ▶ C2S PRO

## Dental Prosthesis Cleaning System



**PRATICIMA**  
healthcare solutions

## ► Oral hygiene

# is essential for a healthy mouth and breath

---

Dentures are rarely cleaned correctly by their owners or by nursing staff.

This is a common problem in retirement homes.

Dentures require specific, regular and meticulous upkeep that is difficult to perform every day.



### The consequences should not be underestimated

Failure to clean and maintain dentures can increase the risk of problems:



#### ON CHEWING

Sensitivity, pain, etc



#### ON HEALTH

Malnutrition, respiratory tract infection



#### ON NUMEROUS ILLNESSES OF THE ORAL MUCOUS MEMBRANE


Irritation, thrush, increased levels of anaerobic bacteria, pain, etc.


## ▶ A cleaning system

# Simple and effective for ensuring the long life of dentures

The C2S Pro cleaning system was developed to improve dental care in healthcare facilities. It significantly lowers bacteria and fungi (candidiasis) levels, reducing the occurrence of some systemic diseases (upper respiratory disease). Regular care of dental prostheses improves the general health of residents. In addition, daily use of the C2S Pro reduces referrals to specialists (dentists, physicians and prosthodontists).

The C2S Pro cleaning system, developed for daily and restorative cleaning of all types of removable and screw-retained dentures through dynamic brushing with special alloy bristles, helps clean dentures thoroughly and prevents the formation of plaque and tartar.

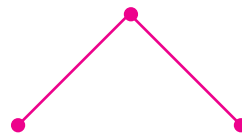

Effect of patented cleaning system  
Standard 



## The C2S PRO eliminates:



**Food**  
particles  
encrusted  
in dentures



**Plaque**  
that naturally  
lines dentures



**Bacteria**  
that are often  
responsible  
for bad breath

## Why use C2S PRO system?

- + Saves time
- + Makes cleaning easier
- + Resident comfort
- + Optimal denture hygiene
- + Ease of use
- + Individual cleaning
- + Standardisation of quality of care



## What are the results on the dental prosthesis?

- Cleans effectively in 2 minutes
- Removes stains
- Polishes
- Prevents risky movements
- Makes dentures look new again



## Technical characteristics

- Dimensions: L. 20 x W. 17,5 x H. 19,5 cm
- Cleaning operation: magnets rotating at 1800 rpm
- Timing: 2 min / 5 min / 10 min / 15 min / 20 min
- Automatic system shutdown
- Certification: CE
- Power supply: separate CE-certified transformer, input: 100–240 V / output: 12 V
- Warranty: 2 years

Reference: 2CPRO



## Two usage protocols



### DAILY PROTOCOL

Usage: cleaning  
Utilisation: daily  
Cleaning time : 2 minutes  
Product to use: C2S Tab effervescent sodium bicarbonate tablet



### MEDICATED PROTOCOL *(in consultation with coordinating physician)*


Usage: healthcare cleaning  
Utilisation: ponctual  
Cleaning time: 10 minutes  
Product to use: pure chlorhexidine

## ► Removable battery

### C2S BAT

---

The C2S Bat battery makes the C2S Pro mobile. The long battery life allows daily use of the C2S Pro cleaning system for a week without recharging (based on daily use of about 1 hour, or 30 two-minute cleanings).

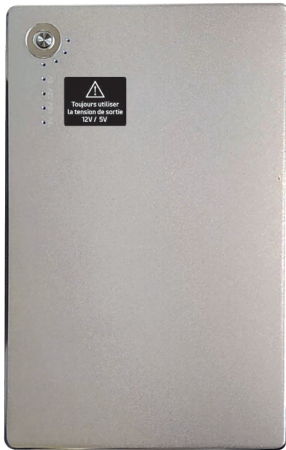

#### Technical characteristics

- Type: Lithium Polymer
- Dimensions: 19,4 x 12 x 2,1 cm
- Colour: silver grey
- Indicators: blue LED
- Autonomy: 7 h
- INPUT power supply: 19 V / 1,2 A
- OUTPUT power supply:
  - DC 12 V / 2,5 A (for C2S system)
  - DC 16 V / 4 A
  - DC 19 V / 4 A
  - USB 5 V / 1,2 A
- Capacity: 30000 mAh

Reference : 2CPROBAT

## ► Cleaning bowl

---

The C2S Pro cleaning system includes a “cleaning bowl”. It is personalised and replaceable (after 6 months) and provides optimal hygiene, improving user comfort while preventing cross-contamination.



#### Technical characteristics

- Dimensions : Ø 11 x H. 9,5 cm
- Certification: CE
- Magenta pink lid

#### Conditioning

Batch of 2 bowls delivered with 2 name labels, 2 label protectors and 2 blisters of alloy bristles

Reference: 2CBOLRM

## ► Bicarbonate tablets

### C2S TAB

---

Designed by C2S and manufactured in France in partnership with ADP Laboratoire, C2S Tab clean deeply in just 2 minutes. They are used daily with the C2S Pro cleaning system. Their active ingredients — sodium bicarbonate and cider vinegar — act as antibacterial and descaling agents. The thyme-mint fragrance leaves breath clean and fresh.



#### Composition

- Sodium bicarbonate
- Cider vinegar
- Thyme-mint fragrance
- Magnesium stearate
- Microcrystalline cellulose
- Blue colouring (E133)

#### Conditioning

200 effervescent sodium bicarbonate tablets

*Reference: 2CTAB*

## ► Dental care trolley

---

The dental care trolley is designed to optimize the efficiency and quality of all dental care in health facilities. It can be equipped with 5 drawers for the storage of prostheses cleaning bowls. Thanks to its standard lateral rails which are compatible with many available accessories, this trolley is easily customizable.



#### Technical characteristics

- Dim.: L. 61.5 x W. 46.8 x H. 98.8 cm
- Natural anodised structure
- ABS side wall, coloured body
- Work surface with rounded raised edges made of thermoformed plastic
- Rolling shutter and code lock
- 4 swivelling wheels Ø 100 mm, inc. 1 with brake
- 1 handle
- 3 standard rails for accessories (2 rails on the right side and 1 rear rail)
- Delivered without equipment

*Reference: 8Y144.DT*



# PRATICIMA

healthcare solutions

---

Rue du Pou du Ciel  
ZI de Reyrieux – CS20129 Reyrieux  
01600 TRÉVOUX – FRANCE

Tél : 04 74 00 98 11  
Fax : 04 74 00 98 36  
E-mail : [praticima@praticima.fr](mailto:praticima@praticima.fr)

[www.praticima.fr](http://www.praticima.fr)